


GYKO PRO

Quick Installation Guide

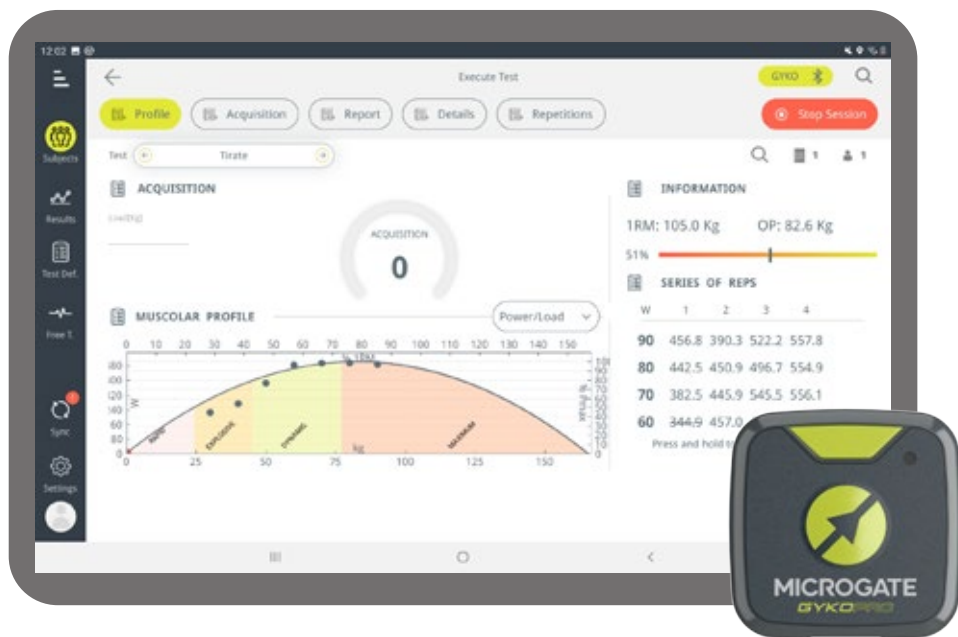

Google Play

App Store




ProMove is available for
Tablets and Smartphones

Watch the Tutorials here
<https://tutorials.microgate.it/>

Stand-alone use with ProMove APP

To use the APP, an account must be created: once registered, a confirmation e-mail will be sent to the registered address. If you have received a coupon, you can activate it directly from the web (<https://www.pro-move.it/>) - otherwise you can choose the subscription plan that suits you.

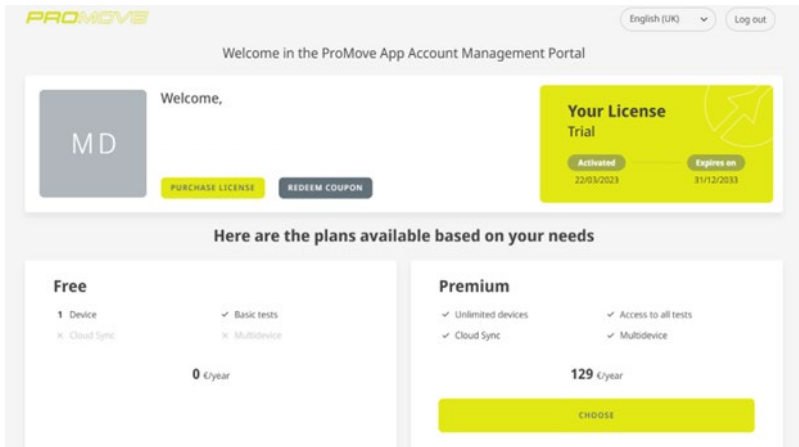



Step 1:

Activate Bluetooth on your mobile device, as well as WiFi and geolocation.


Start the ProMove APP.

In the meantime, you can switch on the GykoPro, simply by pressing the dedicated button. The green light will confirm that it has been switched on.

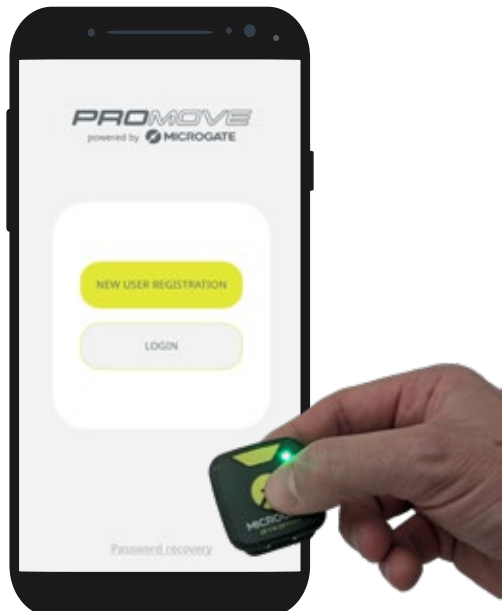

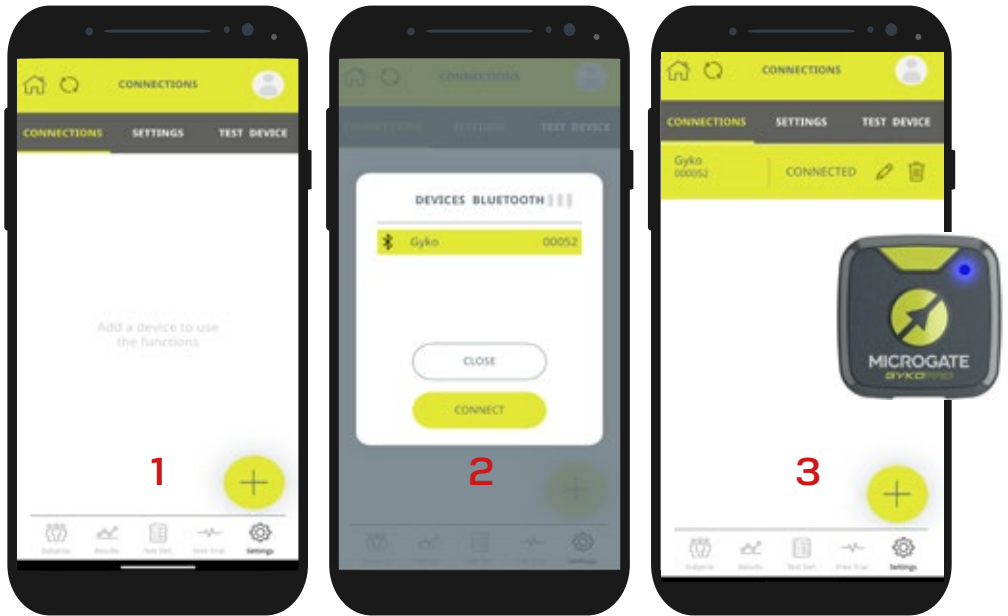



Step 2:

1. Select the Settings menu and use  to search for the device
2. The device will be added to the list - simply select it to activate it
3. If connected, the line on the APP will be coloured green and the device light will start flashing blue.


Step 3:

Once you have defined a test (menu Test Definition ) and entered at least one subject (menu Subjects ), you can use the device.

The next time you access the APP, the previously associated GykoPro will connect automatically.

To switch the device off, simply press and hold the power button.

Note: After a few seconds of inactivity, the device goes into auto-sleep and the LED goes out, in order to ensure a longer battery life.

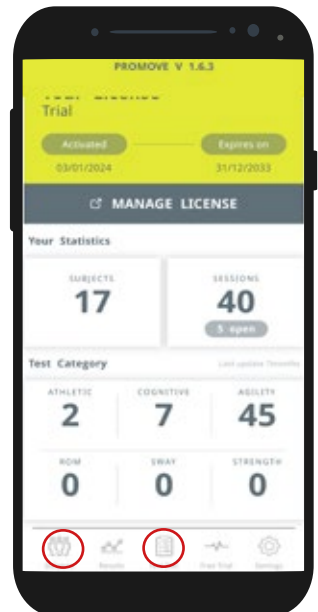

Use in combination with OptoJumpNext or OptoGait


Make sure you have installed the OptoGait/OptoJumpNext software on your PC and updated it to the latest version. The latest version of our software is always available at: <https://medical.microgate.it>

Step 1:

Switch on the GykoPro by holding down the central button

Step 2:

1. Start the software by clicking the OptoGait/OptoJumpNext icon on your desktop.
2. Click on the menu Utility > Devices
3. Click on "New Search" in the bottom right-hand corner
4. Select the GykoPro device from the list
5. Then type the "Activate" button

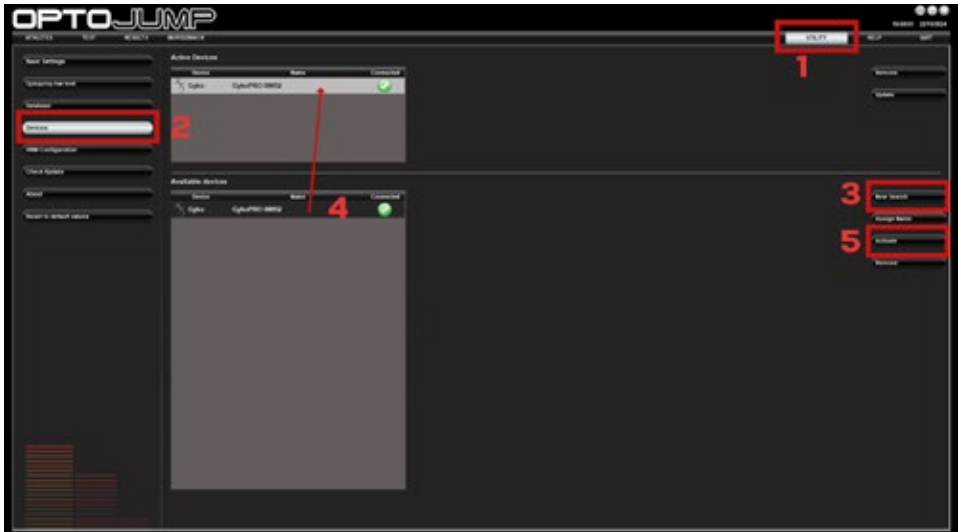

You can now start using the GykoPro inertial sensor with OptoGait and/or OptoJumpNext.

Take a look at our
interactive quick start guide

